



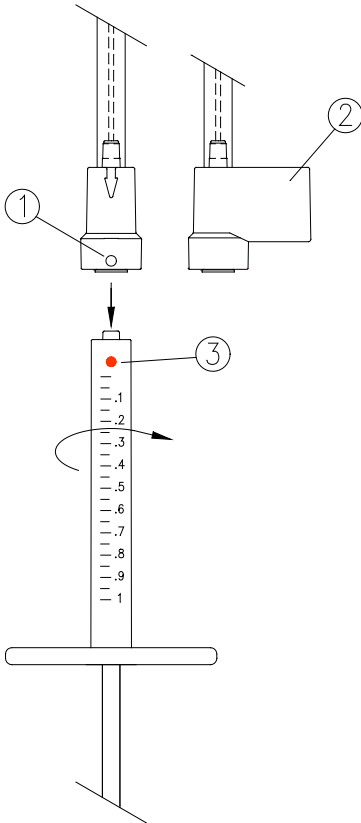

INJECTION NEEDLE

Injection Needle-Syringe Assembly Instructions

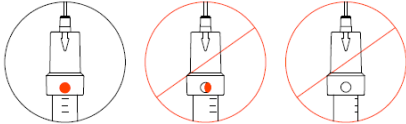

IN00260-00/ SEPT 2024

Rx ONLY Federal law (USA) restricts this device to sale by or on order of a physician.

FEATURES

- The Merz North America needle has a viewing hole or ring (1) on the hub to indicate a correct connection when it is assembled to a Merz North America product syringe (see illustration).
- Single use only. Do not re-sterilize.

ASSEMBLE TO SYRINGE

- Use proper sterile technique when assembling the syringe to the needle.
- If the syringe contents should get on the Luer lock fitting(s), wipe clean with sterile gauze.
- Grasp and hold the needle assembly by its handle (2) with one hand.
- With your other hand, place the Merz North America product syringe onto the needle Luer and turn the syringe until a colored dot (3) is in full view through the needle viewing hole or is aligned with the ring. Do not turn beyond the colored dot. **Be sure the colored dot is aligned.**
- Grasp and hold the syringe and needle assembly with one hand. With your other hand, grasp the mid-section (or handle if present) of the needle shield and pull it directly away from the needle to avoid damaging the needle point or incurring an accidental needle stick.
- Damage or bending of the needle point could cause weakening and/or separation of the needle point from the needle body.

ADDITIONAL SYRINGES

- **Needles With Attached Handle:** To use additional syringes, remove the syringe by grasping and holding the needle by its handle with one hand. With your other hand, grasp and turn the syringe counter to the direction in which you assembled it and remove it from the needle. Attach the new syringe following the steps above.
- **Needles With Removable Handle:** Additional syringes should not be used with these needles.

IMPORTANT SAFETY INFORMATION

- Do not attempt to straighten angled or bent needles.
- Use needle once and discard in accordance with local requirements and safety standards.
- Follow National Safety Guidelines for handling of sharps (needles) to avoid accidental needle sticks with used needles which could transmit HIV (AIDS), HBV (Hepatitis) and other infectious diseases.
- Do not use if package is opened or damaged.
- Do not use if labeling is incomplete or illegible.

Do not use if package is damaged.

WARNING

Contents supplied STERILE using an Ethylene Oxide sterilization process. Do not use if sterile barrier is damaged. If damage is found, call your Merz Pharmaceuticals, LLC representative at 1-855-4MERZTX (855-463-7989).

For single use only. Do not reuse, reprocess, or resterilize. Reuse, reprocessing, or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness, or death. Reuse, reprocessing, or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death of the patient.

After use, dispose of product and packaging in accordance with hospital, administrative, and/or local government policy.

WARRANTY

Merz Pharmaceuticals, LLC warrants that reasonable care has been used in the design and manufacture of this instrument. **This warranty is in lieu of and excludes all other warranties not expressly set forth herein, whether express or implied by operation of law or otherwise, including, but not limited to, any implied warranties of merchantability or fitness for a particular purpose.** Handling, storage, cleaning and sterilization of this instrument as well as other factors relating to the patient, diagnosis, treatment, surgical procedures and other matters beyond Merz Pharmaceuticals, LLC's control directly affect the instrument and the results obtained from its use. Merz Pharmaceuticals, LLC's obligation under this warranty is limited to the replacement of this instrument and Merz Pharmaceuticals, LLC shall not be liable for any incidental or consequential loss, damage or expense directly or indirectly arising from the use of this instrument. Merz Pharmaceuticals, LLC neither assumes, nor authorizes any other person to assume for it, any other or additional liability or responsibility in connection with this instrument. **Merz Pharmaceuticals, LLC assumes no liability with respect to instruments reused, reprocessed or resterilized and makes no warranties, express or implied, including but not limited to merchantability or fitness for a particular purpose, with respect to such instruments.**

Manufactured by:



Merz North America, Inc.
4133 Courtney St., Suite 10
Franksville, WI 53126

Distributed by:

Merz Pharmaceuticals, LLC
6601 Six Forks Road
Suite 430
Raleigh, NC 27615